



Aluminum Modular Ramp User's Guide

WARNING!

Read and follow this user's guide before use.
Failure to follow this guide could result in serious injury or death!

Ramp Capacity: 850 pounds

The combination of the user, the user's mobility device, any assistants, helpers, other people, snow, or any other items on the ramp **MUST NOT** exceed the ramp's capacity.

Caution: Your ramp may be slippery when wet.

Your ramp was designed and installed for proper water drainage.
Water buildup can still occur. Use care and caution when traversing a wet ramp.

Caution should be used at all times.
Proper maintenance and upkeep to the ramp surface is vital.

Proper Ramp Use

Do not use the ramp if it is damaged or unstable. If any part of the ramp is damaged or loose, **DO NOT USE UNTIL REPAIRS ARE MADE BY A CERTIFIED INSTALLER OR OTHER QUALIFIED PERSON.**

- Do not play on or near the ramp.
- Use only with the assistance of a qualified helper.
- Always use a lap belt.
- Refer to your mobility device Owner's Guide for the proper degree of incline/decline and chair direction before attempting ramp use. Never exceed its recommendations.
- Always use the handrails.
- Make smooth, slow turns and transitions making sure all items are clear of any corners or handrails.
- Avoid stopping and starting on any sloped part of the ramp.
- Do not move or adjust your ramp without the help of a qualified ramp installer.
- Keep the ramp clean and clear of debris and other items.

Proper Ramp Maintenance

- Have the ramp inspected and adjusted by a qualified ramp installer annually to ensure the highest level of performance and safety.
- Sweep the ramp with a broom as needed, to remove all dirt, leaves, or any other debris.
- Keep your ramp clear of snow and ice. **DO NOT** use salt or salt-based ice melt products. **A magnesium chloride ice melt is recommended.**
- Wash, rinse, and dry the ramp annually. Wash with a soft brush, mild soap, and water. (Avoid alkaloid detergents) Spring is the recommended time to wash the ramp. Especially if the ramp was subject to snow, ice, and magnesium chloride ice melt products. Drying the ramp is recommended to remove any excess solution and water and to restore the ramp's original grip and traction on railings and surfaces.

If you have any additional questions or concerns please contact us.

Thank you for choosing



269.341.3100

info@liomobility.com

www.liomobility.com